



Volk Optical Inc.
7893 Enterprise Drive
Mentor, OH 44060 USA
Tel: 440-942-6161
Fax: 440-942-2257
Email: volk@volk.com

EC REP

EU Representative:
Rudolf Riester GmbH
Bruckstraße 31
72417 Jungingen, Germany
Email: info@riester.de
Phone: +49 74 77 / 92 70-0
Fax: +49 74 77 / 92 70-70



Volk®1 Single Use Vitrectomy Lenses

- STERILE. STERILIZED WITH ETHYLENE OXIDE GAS.
- FOR SINGLE USE ONLY.
- DO NOT USE IF THE PACKAGE IS OPEN OR DAMAGED.
- FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

ENGLISH: INSTRUCTIONS FOR USE

INDICATIONS FOR USE

The Volk®1 Single Use Vitrectomy Lenses are indicated for use as diagnostic lenses for eye fundus examinations and use in the therapy of intraocular abnormalities. The devices are sterile, single use, disposable lens systems.

SPECIFICATIONS

Product	Product Code	Magnification	Field of View
Volk®1 Single Use Flat Vitrectomy	VFD	1.00x	36°
Volk®1 Single Use 1.5x Magnifying Vitrectomy	VMD	1.50x	30°
Volk®1 Single Use Wide Field Vitrectomy	VWFD	0.50x	48°
Volk®1 Single Use 30° Prism Vitrectomy	V30PD	0.80x	33° (offset 30°)
Volk®1 Single Use Biconcave Vitrectomy	VBCC	1.00x	25°
Volk®1 Single Use Flat SSV	VFLATSSVD	0.92x	30°



WARNINGS

1. Do not immerse the device in a decontaminating product before use.
2. Inspect the contacting surface(s) to make sure it is free from any damage (e.g. chips, scratches, etc.). Do not use or repair if the device is damaged.



PRECAUTIONS

1. Do not attempt to use the Volk®1 Single Use Vitrectomy Lens prior to completely reading and understanding these instructions for use.
2. To be used by a licensed physician in a method consistent with other direct image ophthalmic vitrectomy lenses.
3. The sterile packaging and lens should be inspected prior to use.
4. The Volk®1 Single Use Vitrectomy Lens is intended for single patient use only.
5. Do not resterilize and reuse.
6. This device is packaged and sterilized for single use only. Do not reuse, reprocess, or resterilize. Reuse, reprocessing, or resterilization may create a risk of contamination and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.

"USE BY" DATE

Use the device prior to the "Use By" date on the package label.

SUGGESTED DIRECTIONS FOR USE

1. Remove the lens from its package and place it in a sterile work area.
2. All lens designs require methylcellulose or other similar viscous interface solution be applied to the concave contact surface (i.e., between the cornea and the device).
3. The Volk®1 Single Use Vitrectomy Lens is a surgical lens device that helps improve accuracy and clarity of images of the posterior and peripheral retina and of the vitreous body during surgical intervention. The lens is installed in the cornea and helps increase magnification.

STERILE EO

Sterilized using ethylene oxide



Do not re-use



Consult the Instructions for Use for important cautionary information



Do not use if package is damaged or open



Use-by date

LOT

Lot number

REF

Reference number



Manufacturer

EC REP

Authorized representative in the European Community