

THIS GUIDE APPLIES TO THE FOLLOWING DEVICES ONLY

VOLK PORTABLE CAMERAS		
Volk Pictor Prestige	Volk VistaView	Volk Viva

The information contained within this document applies only to Volk's portable cameras (see above). Information regarding the care of other devices can be found on Volk's website: www.volk.com

STORAGE

Ensure that devices have been cleaned, disinfected (if applicable), and dried before storage. Store devices in a clean, dry, room-temperature environment in their appropriate, supplied case.

CLEANING & DISINFECTION

CLEANING & DISINFECTION STEPS

Volk Pictor Cameras

Handset, Module, Cradle

1. Shut down the handset and remove the cradle from the main power source.
2. Disinfect the handset housing and cradle with an alcohol wipe (70% alcohol). Avoid contact with the system connectors.
3. Clean the optical surfaces with a soft, lint-free cotton cloth

Silicone Eye cup

1. Disinfect the eye cup using an alcohol wipe (70% alcohol), or soak the eye cup in a glutaraldehyde, hydrogen peroxide, or peracetic acid solution.
2. Rinse with eye cup under running water.
3. Dry the eye cup using a clean cloth before use.

NOTES

- + The Pictor Prestige is not meant to be sterilized.
- + The eye cup should be disinfected before each use with a new patient.
- + Do not immerse the handset, module, or cradle in any fluids.

VistaView

1. Ensure that the camera is turned off and disconnected from any external power sources.
2. Clean the external surface with a clean, soft, non-shedding cotton cloth dampened with a detergent/water solution (2% detergent by volume). To prevent fluids from entering the device, do not spray the device with water/detergent solution. Avoid touching the electrical connectors and optical lens. Hand dry using a clean, non-shedding, cotton cloth.
3. The primary imaging lens can be cleaned using Volk Precision Optical Lens Cleaner.
4. Clean the eye cup using a non-shedding cotton cloth dampened with a 70% ethanol solution.

NOTES

- + The Volk VistaView is not meant to be sterilized.
- + The eye cup should be disinfected after each patient exam.
- + Do not immerse the camera in any fluids.

Viva

1. Ensure that the camera is turned off and disconnected from any external power sources.
2. Clean the external surface with a clean, soft, non-shedding cotton cloth dampened with a detergent/water solution (2% detergent by volume) or water/isopropyl alcohol solution (70% IPA by volume). To prevent fluids from entering the device, do not spray the device with water/detergent solution. Avoid touching the electrical connectors and optical lens. Hand dry using a clean, non-shedding, cotton cloth.
3. The primary imaging lens can be cleaned using Volk Precision Optical Lens Cleaner.
4. Clean the eye cup using an alcohol wipe (70% alcohol), or soak the eye cup in a glutaraldehyde, hydrogen peroxide, or peracetic acid solution. Rinse under running water. Dry using a clean cloth before use.

NOTES

- + The Volk Viva is not meant to be sterilized.
- + The eye cup should be disinfected after each patient exam.
- + Do not immerse the camera in any fluids.

Wiping the optical surfaces with a microfiber cloth will cause lens damage. Be sure to use only a soft, lint-free cotton cloth.

DISCLAIMERS

- + Only follow the reprocessing procedures listed in Volk's Instructions for Use, and as stipulated by your hospital/facility.
- + Always ensure proper regulatory compliance from your competent authority and facility when choosing a reprocessing technique (FDA, DGHM, etc.).
- + The instructions provided within have been validated by Volk Optical as being CAPABLE of preparing a medical device for re-use. It remains the responsibility of the processor to ensure that the reprocessing as actually performed using equipment, materials and personnel in the reprocessing facility achieve the desired result. This normally requires validation and routine monitoring of the process.