




Locating the License Key in Pictor Workstation

Please follow the steps below to learn how to find the license key for your Pictor Workstation software.

1. Launch Pictor Workstation on your computer
2. Select the information tab located in the upper-right portion of the screen

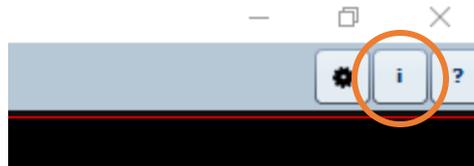

3. Select "Register"
4. You will be able to view and copy your license key in the registration window.
 - a. The key will have the following format: XXXXX-XXXXX-XXXXX-XXXXX

You have now successfully located your Pictor Workstation license key! If you need any additional assistance, please contact customer support at support@volk.com, or call us at 440-942-6161.